

THE CRADLE SPLINT

for the foot and ankle

Instructions for use

- Use of the splint must be according to the instructions given by the treating physician.
- We recommend not to apply the splint directly to the bare skin.
- Prior to applying the splint all skin wounds or lesions should be dressed.
- When the splint is being inflated not with the attached air pump, the air pressure in the splint should not exceed 230 mmHg (30,000 pascal).
- The splint can be washed with water, soap, and the usual antiseptics in use in medical institutions.
- Do not launder in a washing machine.
- Sterilize the splint by gas when necessary.
- Do not open the package with a sharp instrument.
- Do not puncture the splint.
- Store at temperatures below 50 degrees Celsius.
- The splint is intended for personal use.

The company takes no responsibility if the splint is not used according to the instructions given by the producer.

Distributor: Align Orthotics.
PO Box 408
Malvern, VIC 3144
T +61 3 9822 2008
F +61 3 9923 6260
E info@alignorthotics.com.au
W www.alignorthotics.com.au



- Cross straps
- Oblique straps
- One-way valve
- Buckles
- Pocket for stabilizer



